

AURIX™

AURIXSYSTEM.COM

HEAL ON.

CARING FOR YOUR WOUND AFTER AURIX™

**HARNESS YOUR
NATURAL
HEALING POWER**

**YOU ARE
THE SECRET
INGREDIENT**



Nuo Therapeutics, Inc.
207A Perry Parkway, Suite 1
Gaithersburg, MD 20877

Phone 1.866.298.6633
Fax 1.240.499.2690
www.nuot.com

WHAT IS AURIX™?

Skin ulcers and wounds that don't heal can have a significant impact on your quality of life and present a treatment challenge for healthcare providers. Aurix™ is a biodynamic hematogel that harnesses your natural healing mechanisms in order to jumpstart the healing process from deep within the wound bed.

MANAGING YOUR WOUND CARE WITH AURIX™

After treatment with Aurix, your wound care team will apply dressings to your wound. The following information will help you manage your wound and put you on the path to healing.

WOUND COMPLICATIONS

Complications may occur while your wound is healing. One complication that may occur is infection. After your visit with your doctor, it is important to inspect the visible wound area for infection every day.

HOW TO INSPECT FOR INFECTION

1. Does the area around the dressing look red or swollen?
2. Is there a large amount of fluid or pus seeping around the edges of the dressing? (Non-odorous drainage is normal.)
3. Is the area around the dressing hot when touched?
4. Does the dressing have a bad or sickly smell?
5. Does the skin around the dressing look white and feel wet?
6. Does the wound appear to have extended beyond the dressing?

If you answered "yes" to two or more of these statements, please call your study doctor for an immediate appointment.

KEEP PRESSURE OFF YOUR WOUND

Placing pressure on your wound can kill the newly forming cells and slow down your healing. It is very important to keep all pressure or weight off your wound at all times, even for a short walk to the bathroom.

PRESSURE-RELIEVING DEVICES

Keeping all pressure off your wound is the most important thing you can do to help the healing process.

Your wound care team may give you a special boot or other device to help relieve the pressure on your wound. You may be given crutches, a walker, or a wheelchair to help with walking. Regardless of the mechanism, make sure to use a pressure-relieving device at all times.

It is important to keep your dressing and offloading device dry .

BATHING

Bathing or showering is permitted, but it is important to keep all dressings dry.

When taking a shower, make sure to completely cover your device with a plastic bag.

Secure the bag to your skin with waterproof tape or a coban wrap.

Do not use very hot water or harsh cleansing agents around the dressing.

If your secondary dressing (top layer) gets excessively wet, you must replace it.

GET YOUR QUESTIONS ANSWERED

Your wound care team is here to help your wound heal as quickly as possible. They can help you so that your wound care treatments become a part of your everyday life. Make sure to ask questions and attend all appointments with your wound care team.

www.AurixSystem.com

Keep your wound on the path to healing.
Make sure to keep all office and clinic visits.